

Disposable Respirators Seal Checks Infographic



User seal checks

Disposable Respirators

NEGATIVE AND POSITIVE PRESSURE



Cover respirator with both hands

Breathe **in** gently

Respirator should slightly **collapse**

If air leaks **in** around the face:



Adjust straps or
Reposition face piece



Cover respirator with both hands

Breathe **out** gently

Respirator should slightly **expand**

If air leaks **out** around the face:



REPEAT TEST UNTIL SEAL IS ACHIEVED

Source: safetyalliancebc.ca/covid-19